

Features	 LIFEPAK CR2 (W-Fi or 4G)	 LIFEPAK 1000	 HeartSine 350/360P	 ZOLL AED 3	 ZOLL AED PLUS
Device Type	Semi-automatic/ Fully-automatic	Semi-automatic	Semi-automatic/ Fully-automatic	Semi-automatic/ Fully-automatic	Semi-automatic/ Fully-automatic
CPR Guidance	Yes, it provides real-time voice guidance with instructions like “Place the heel of your hand in the center of the chest” Has a built-in metronome, a ticking sound, to keep compressions on beat	Yes, visual & audible prompts	Yes, visual and audible prompts guide your CPR, with a built-in metronome	Yes, Real CPR Help™ provides live feedback like “Push deeper” or “Slow down” Has a built-in metronome to guide compression rate	Yes, Real CPR Help™ provides live feedback like “Push deeper” or “Slow down” Has a built-in metronome to guide compression rate
Device Readiness Indicator	Green flashing light; Beeps when detects issues; Wi-Fi versions can also be tracked from a centralized software system	OK symbol when AED is ready for use. If it encounters an issue, a wrench will appear	Green flashing light; Beeps when detects issues	Green tick; changes to a red X when detects issues	Green tick; changes to a red X when detects issues
Shock Advisory	Real-time visual & audio	Real-time visual & audio	Real-time visual & audio	Real-time visual & audio	Real-time visual & audio
Max Shock Energy	360J (Adult/ 90J (Pediatric)	360J (Adult)/ 90J (Pediatric)	200J (Adult)/ 50J (Pediatric)	200J (Adult)/ 85J (Pediatric)	200J (Adult)/ 85J (Pediatric)
Escalating Energy	Yes	Yes	Yes	Yes	Yes
Pediatric Mode	Built-in child mode button	No, pediatric pads required	No, pediatric pads required	Built-in child mode button	No, pediatric pads required
Electrodes	Universal Adult/ Child pads	Adult and Pediatric Pads sold separately	Adult and Pediatric Pad-Paks (battery & electrodes combined)	Universal Adult/ Child pads	Adult and Pediatric Pads sold separately
Battery Life	4 years standby	4 years standby	4 years standby	5-year standby (approx. 3 years with daily self-checks enabled)	5-year standby (approx. 3 years with daily self-checks enabled)
Electrode Life	4 years; 5 years	2 years	4 years	5 years	5 years (adult pads); 2 years (infant pads)
Self-Testing	Daily/weekly/ monthly self-checks	Daily	Weekly self-test	Automatic self-tests (weekly/monthly; daily option available but reduces battery life)	Automatic self-tests (weekly/monthly; daily option available but reduces battery life)
Connectivity	Wi-Fi & cellular LIFELINKcentral™ is the free software to track all AEDs	No connectivity	No connectivity	Wi-Fi enabled with ZOLL PlusTrac™ software to track AEDs (available at additional cost)	Wi-Fi enabled with ZOLL PlusTrac™ software to track AEDs (available at additional cost)

Features	 LIFEPAK CR2 (W-Fi or 4G)	 LIFEPAK 1000	 HeartSine 350/360P	 ZOLL AED3	 ZOLL AED PLUS
IP Rating (resistance to moisture & dust)	IP55	IP55	IP55	IP55	IP55
Weight	~2.0 kg (4.5 lbs)	~3.2 kg (7.1 lbs)	~1.1 kg (2.4 lbs)	~2.5 kg (5.5 lbs)	~3.1 kg (6.7 lbs)
Dimensions (L×W×H)	8.9" × 9.2" × 3.5"	10.9" × 9.2" × 3.4"	8.0" × 7.25" × 2.0"	9.2" × 9.9" × 3.2"	11.5" × 9.5" × 5.25"
Voice/Visual Prompts	Clear voice & visual prompts	Voice prompts & visual prompts	Voice prompts, LED indicators	Voice prompts, visual icons, CPR bar display	Voice prompts & visual prompts
Warranty	8 years	5 years	8 years	8 years	8 years
Data Transfer	USB, Wi-Fi, Cellular	IrDA cable	Requires data cable	USB, Wi-Fi	USB, Wi-Fi
cprINSIGHT™ (continuous compressions during analysis for less CPR interruptions)	Yes	No	No	No	No
ClearVoice Technology; automatically adjusts the volume prompts based on ambient background noise	Yes	No	No	No	No
Compatible with BC Fire and Emergency Medical Services	Yes, electrodes are compatible with Fire LP1000 & British Columbia Emergency Health Services (BCEHS) LIFEPAK 15	Yes	No	No	No
LCD Screen	No	Yes	No	Yes	Yes
GPS Tracking	Yes, Wi-Fi tracking when in range; 4G can always be tracked	No	No	No	No